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
- *Dust separators mainly for dental technician use*

#### Available models


**Item  
090077**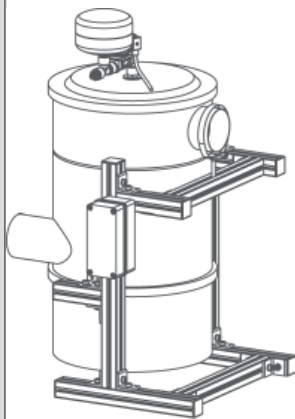

**DUST SEPARATOR WITH PNEUMATIC AND  
AUTOMATIC CLEANING OF THE FILTER \***

Suggested for:  
Tecno-Jet and Flux-Jet units.  
Max. air flow 300 m<sup>3</sup>/h - 2/3 work stations in  
aspiration.  
W= 580 mm D= 550 mm H= 900 mm


**Item  
091040**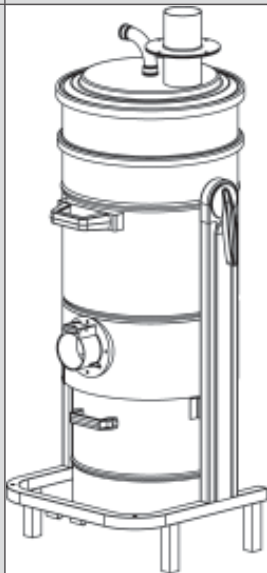

**35 LITRE DUST SEPARATOR  
PNEUMATIC CLEANING OF THE FILTER \***

The filter cleaning is carried out automatically by  
compressed air.  
Dust collects inside a disposable plastic bag which  
can be easily taken out.  
Max. air flow 700 m<sup>3</sup>/h - 6/7 work stations in  
aspiration.  
W= 505 mm D= 590 mm H= 1275 mm

\*DIN EN 60335-2-69 filters.
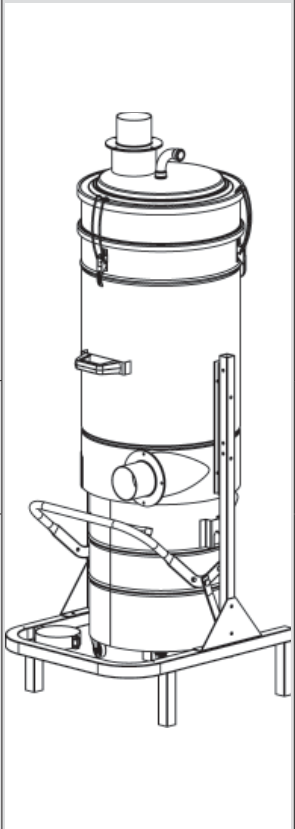
<p><b>Item 091060</b></p>		<p><b>75 LITRE DUST SEPARATOR PNEUMATIC CLEANING OF THE FILTER *</b> The filter cleaning is carried out automatically by compressed air. Dust collects inside a disposable plastic bag which can be easily taken out. Weight: 82 Kg Max. air flow 900 m<sup>3</sup>/h - 9 work stations in aspiration. W= 650 mm D= 700 mm H= 1950 mm</p>
<p><b>Item 091062</b></p>		<p><b>75 LITRE DUST SEPARATOR</b> Max. air flow: 1100 m<sup>3</sup>/h - 12 work stations in aspiration</p>
<p><b>Item 091064</b></p>		<p><b>75 LITRE DUST SEPARATOR</b> Max. air flow: 1500 m<sup>3</sup>/h - 14 work stations in aspiration</p>

ASPIRATION MOTOR	HEAD m <sup>3</sup> /hour	TANK CONN. DIAMETER	N. OF WORK BENCHES
1 UNI-JET 500	500	90	6
2 UNI-JET 500	1000	110	12
3 UNI-JET 500	1500	140	18
1 UNI-JET 1000	1000	110	12
2 UNI-JET 1000	2000	140	24

\*DIN EN 60335-2-69 filters.

- *Introduction*
- *Warranty*
- *Warnings*

- **Introduction**

Dear Sirs,

thank you for the choice of dust separator. We are sure that you will be satisfied with it. It is a high technology product designed to grant long lasting and perfect working. This equipment is manufactured in compliance with the Machinery Directive EMC 89/392/CEE (see the CE label). Please, keep this instruction manual with care and make sure that users read it. In case the original manual got lost you can download the manual from the web site [www.cattani.it](http://www.cattani.it). If you have any suggestion which can be of help for us to improve our products, please do not hesitate to inform us. We reserve the right to make changes to our products and to update manuals without engagement on our part to modify the products previously supplied. For any inquiry, please contact our local authorized technician or Cattani S.p.A. directly.

- **Warranty**

This machine has been tested before delivery to make sure that it is working perfectly. Users cannot alter the machine operation for no reason; in case of anomalies contact our local authorized technician.

The manufacturer is relieved from any liability in case of:

- failure to return the warranty slip to the manufacturer;
- failure to follow instructions;
- use of non-authorized spares parts;
- failure to carry out maintenance interventions.

- **Warnings**

Before starting using the machine read this manual carefully.

Do not aspirate toxic substances, unless you install a special filter (specific for the type of dust) which can be supplied on demand.

Do not aspirate flammable materials or substances with risk of explosion (fuel, solvents, etc.), unless the the machine is fitted with flame-proof motor and electrical system which can be supplied on demand.

It is forbidden to aspirate corrosive substances, unless the machine is fitted with a special container (stainless steel AISI 304).

We discourage from using the machine without the supplied filter.

Do not aspirate moist or encrusting dust.

If the filters are assembled correctly the machine does not throw any object around.

To empty the container of toxic and harmful substances and to clean the filters, use the specific suitable P.P.E.

Do not fill the container above the level shown on the display.

Do not lay objects on horizontal protection covers.

The machine box contains movable parts. Do not remove it.

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- *Installation*
- *Operation*

- **Installation**

- Use a flexible hose with a suitable diameter to connect the tube holder A1-A2 to the aspiration piping coming from the work benches.
- Connect the tube holder B1-B2 to the aspiration group by means of a flexible hose.
- Feed the tube holders C1 and C2 with dry compressed air at a suitable flow.

Suggested air pressure:

C1= 5,5-6 bar / C2= 6,5-8 bar

The tube holder A2 has always a 140 mm Ø diameter.

The tube holder B2 is available in four diameters:

90 mm for 500-900 m<sup>3</sup>/h flow

110 mm for 900-1000 m<sup>3</sup>/h flow

120 mm for 1000-1500 m<sup>3</sup>/h flow

140 mm for 1500 -2000\* m<sup>3</sup>/h

- **Operation**

The aspirated material is passing through the pipe union and it is tangentially deviated to the internal wall of the container by an internal deflector.


Heavy materials collect on the bottom of the container, while the small dust particles are trapped on the filter surface. The filtering media conforms with DIN EN 60335-2-69 appendix AA standards. The filtered air is exhausted from the top of the separator. It is advisable to convey it outside, preferably on rooftops so as to avoid any problem with other activities or houses.

Every 20 minutes of work and even more often if necessary, a blast of counter-current compressed air at 6/8 bar will automatically clean the filter.

In case of plants subject to intensive work with dust, specially in case of very thin dust, as the zirconium dust, in the evening at the end of the working day, or in the morning before starting to work, turn the suction machine and the compressor on, in order to carry out some washing while the plant is not working.


\* for 2000 m<sup>3</sup>/h a longer filter cartridge is necessary

Item 090077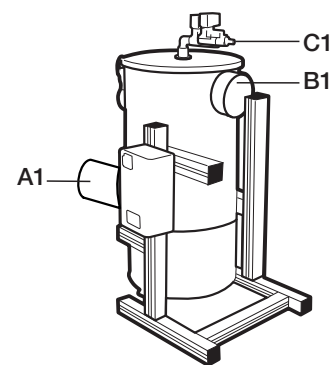

A1-B1 fix Ø80 mm tube holder

Item 091060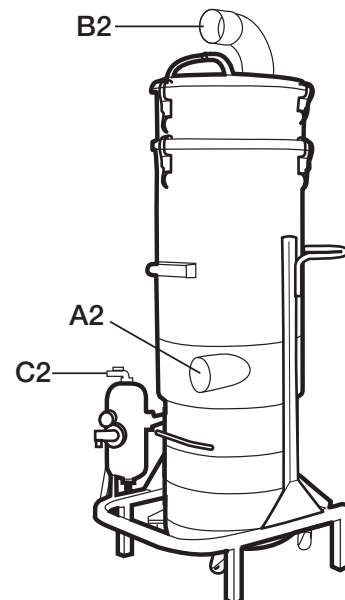

C2 air tank for the automatic rinsing of the filter

- *How to empty the container*
- *Use and maintenance*
- *Basket (ref. item 091060-2-4)*


- **How to empty the container**

Before lowering the dust container release lever, wait until the motor stops and pneumatic cleaning or shaking has been completed.

To remove the container, lift the release lever placed on the chassis. The container can be now easily transported on its wheels.

When the container has been emptied, replace it in its original position and make sure that the slideways are in their seat. Lower the release lever and the container will be automatically locked.

At this point the aspirator is ready to be used again.

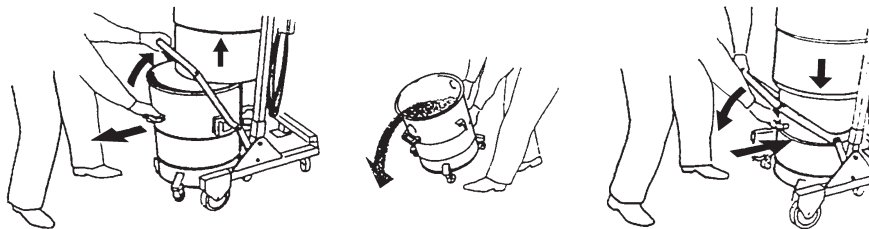- **Use and maintenance**

For the good operation of the machine we suggest to check:

- if the filter is worn or too much permeable;
- if the gaskets of the head seating, of the filter holding ring and of the container are sealed;
- if screws and nuts are tight;
- that the container is not filled to the top.

We suggest to empty it when it has reached 3/4 of its max. capacity.

- **Basket (ref. item 091060-2-4)**

This optional keeps the dust (chalk) and aspirated material collection bag open.



- *How to replace the cartridge filter*  
(ref. item 091060-2-4)

- *How to replace the cartridge filter* (ref. item 091060-2-4)

- 1) release the three clamps on the filtering chamber;
- 2) lift the tank group;
- 3) lift the cartridge filter group;
- 4) unscrew the three handles on the top of the cartridge high side;
- 5) release the cartridge filter;
- 6) carry out the inverse operation for re-assembly.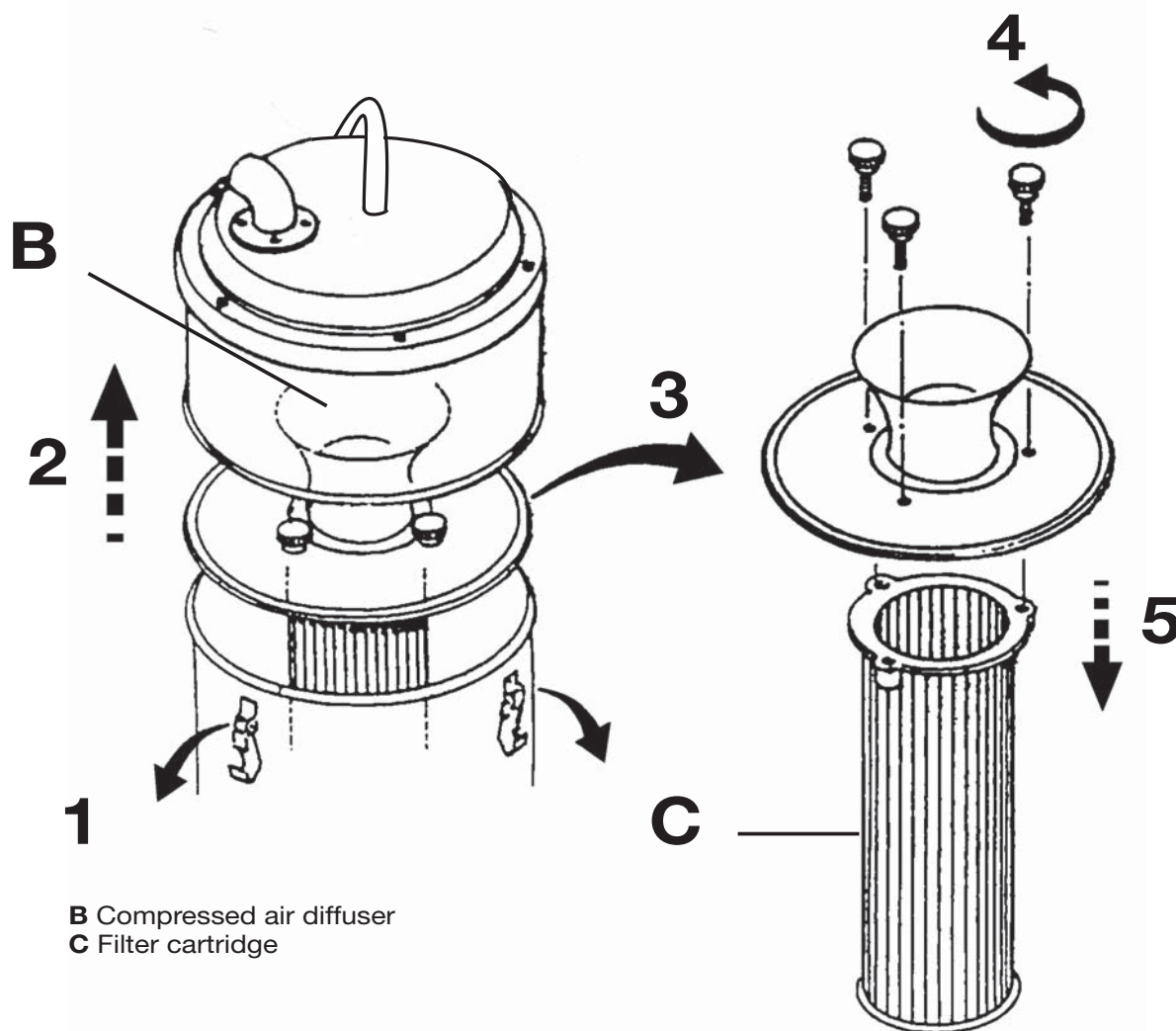

- *Dust separator (ref. item 090077)*

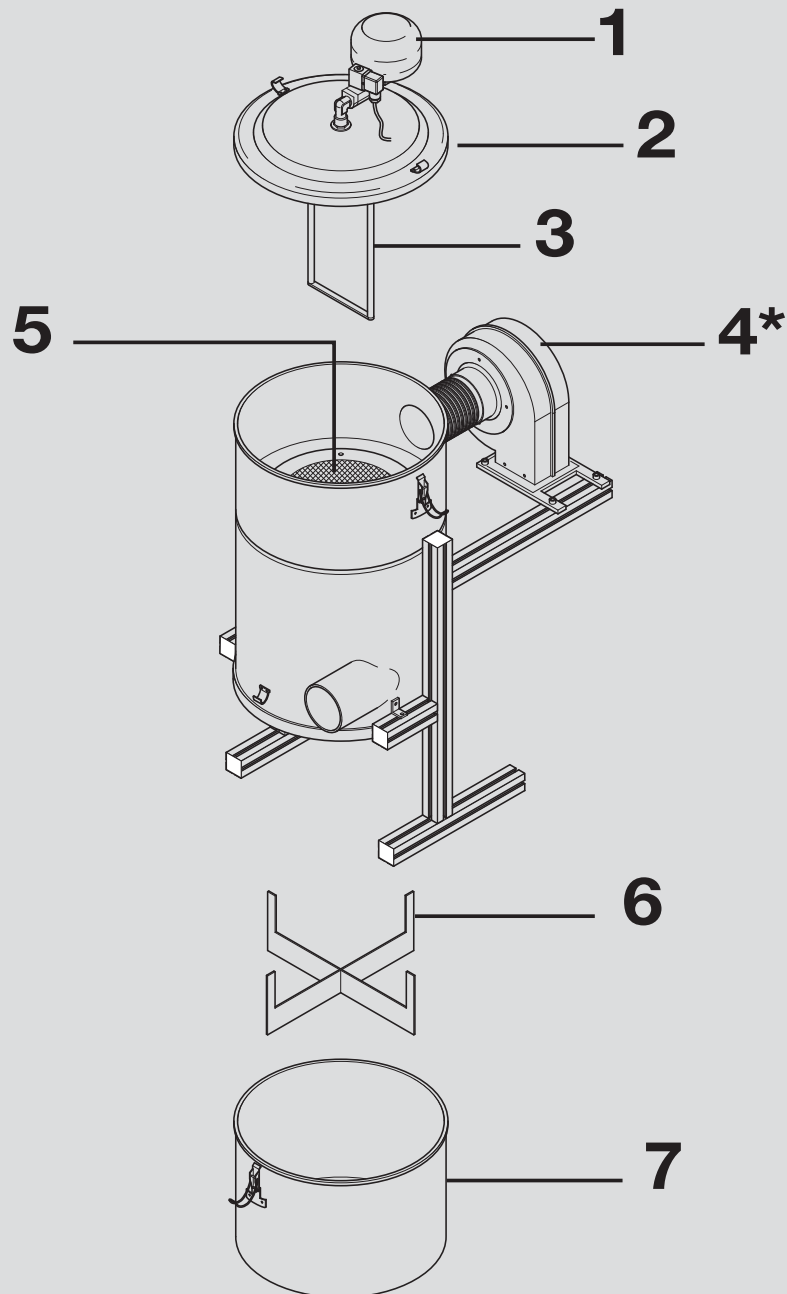- 1 Air tank
- 2 Cover
- 3 Compressed air round diffuser
- 4 Aspiration group
- 5 Filter cartridge
- 6 Bag holder frame
- 7 Dust container

**\* The aspiration group is present in case the separator is working independently, whereas it is not present in case the dust separator is included in a centralized aspiration system.**

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
*In compliance with the rules of fluid dynamics and keeping into consideration the needs regarding energy saving and environmental sustainability it is possible to manufacture long-lasting small and big aspiration systems with reduced working costs.*

*We are at customers' disposal for the design of customized aspiration systems, we only ask their confidence in return.*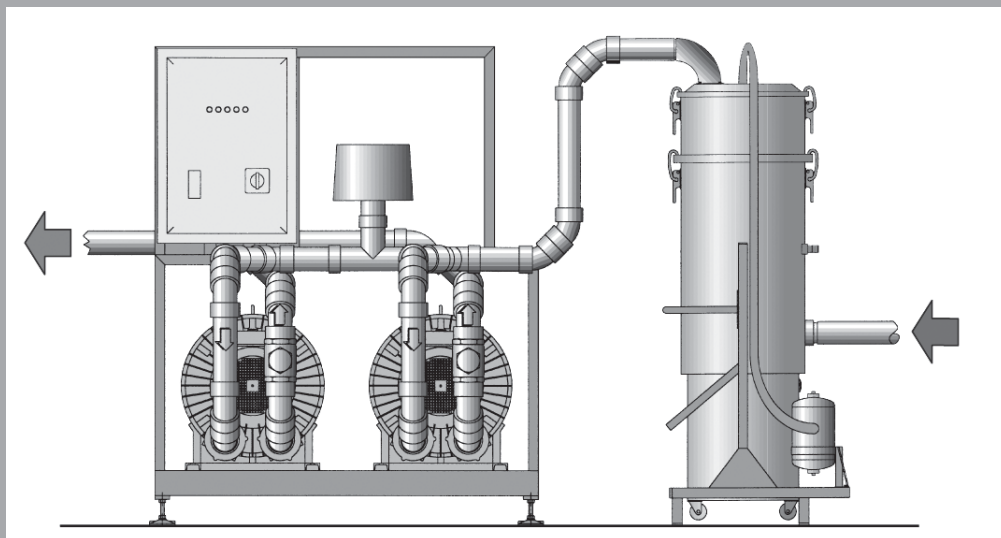

**HOW CAN WE DO  
WE LEAD IN OUR FIELD,  
YET WE COST LESS THAN  
THE ALTERNATIVES?  
THIS IS HOW:**

- **WE RESEARCH:** this lets us apply the latest technology in all of our products and solutions.
- **WE INCREASE PERFORMANCE:** electronic and information technology allow us to increase the performance and reliability of our products.
- **WE REDUCE COSTS:** less maintenance and energy costs mean on a cost-benefit analysis we are always the most economical.
- **WE REDUCE ENVIRONMENTAL IMPACT:** we save 50% of primary materials, and allow you to save between 30% and 50% of electrical consumption.

**C**  
**CATTANI**

**AIR TECHNOLOGY**

6/A Via Natta. 43122 Parma, Italy

10°21'48" EST - 44°50'46" NORD

Ph. +39.0521.607604

Fax +39.0521.607628 (Sales Dept.)

Fax +39.0521.607855 (Purchasing Dept.)

Fax +39.0521.399966 (Accounting Dept.)

www.cattani.it - e-mail: info@cattani.it

Company with Quality System Certified according to  
**UNI EN ISO 9001:2008 - UNI EN ISO 13485:2012**