MAGNOLIA
CHEMICAL PROFESSIONALS

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**CONCENTRATED DISINFECTANT-DETERGENT
FOR DENTAL ASPIRATORS WITH ANTI-SCALE AGENT**

Application: single-surgery and centralized aspiration systems

When to use: At the end of the working day, and possibly half-way through the day, for a detergent-disinfectant wash. After intervention on high infection risk patients

How to use:

- For a cleaning-disinfectant action (20 cc in 2,5 l of water): prepare a solution at 0,8% in hot water (about +50 °C).
- For a cleaning-sanitizing action (10 cc in 2,5 l of water): prepare a solution at 0,4%.
- Suck from the Pulse Cleaner through the equipment. With 1 litre of product it is possible to obtain 250 litres of sanitizing solution at 0,4% and 125 litres of disinfectant solution at 0,8%. Let it work during the night or for at least 1 hour.

Range of action:

- Detergent
 - Anti-scale agent
 - Disinfectant: bactericide, fungicide, tuberculicide - Virucidal (HIV, HBV, HCV)
- Characteristics:** It does not contain phenol. Citrus-scented.

Precautions: Use protective gloves and goggles when preparing the solution. Read carefully the instructions, warnings and SDS before use

Certificates:

- Class IIa-Medical Device
- CE 0425
- The product has been tested to be effective according to the standards:
EN 13727-14561, EN 13624-14562, EN 14563, EN 14476

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**COMPLEMENTARY
PRODUCTS FOR
SUCTION SYSTEMS**

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Magnolia is part of the Cattani group which is
a leader in the production of
dental aspiration systems.
Nobody better than the manufacturer can
suggest suitable disinfectants
for his products.

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SANITIZING DETERGENT FOR THE CLEANING OF DENTAL ASPIRATORS

Application: single-surgery and centralized suction systems

When to use: Normally at the end of the working day and possibly half-way through the day to clean and sanitize the plant.

How to use:

- Prepare a solution at 5% in hot water (about +50 °C).
- Suck from the Pulse Cleaner through the equipment. With 1 litre of product it is possible to obtain 20 litres of solution at 5%.

Range of action:

- Detergent
- Sanitizing-antibacterial (let it work during the night or for at least 1 hour).

Precautions: Use protective gloves and goggles when preparing the solution - Read carefully the instructions, warnings and SDS before use.

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You can obtain a thorough cleaning of the aspiration system with:

- "Pulse Cleaner" creating the necessary turbulence.
- "Puli-Jet Plus New" disinfectant-detergent.
- Antifoam tablets, inhibiting foam formation and reducing the bacterial rate.

"Pulse Cleaner" is a new turbulence-maker. To prepare the Puli-Jet Plus New solution, pour the concentrated product in warm water (+50 °C) and then place the terminals on the pistons. The cleansing aspiration will start without any further operation. The solution left in the container can be used to sanitize the spittoon bowl. Do not rinse the aspirator; do not turn on the spittoon-rinsing tap: Puli-Jet Plus New will go on disinfecting and dissolving blood and mucus all the night long.

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DISINFECTANT ANTIFOAM TABLETS FOR DENTAL ASPIRATORS

Application: single-surgery and centralized suction systems

When to use: During work, every day, after cleaning and disinfection.

How to use: At the end or at the beginning of every working day, after cleaning, place 1 or 2 tablets in the first filter: when aspirated, the liquid will take up the right quantity of disinfecting antifoam product.

Range of action: The effectiveness lasts as an average a full working day. Check every 3/4 hours that the tablet has not melted and place a new one, if necessary

- Antifoam
- Disinfectant: bactericide, candidicide

Precautions:

- Do not remove the protective film: it dissolves in the aspirated liquid.
- For small filters: remove the film, then break the tablet in half by pressing along the pre-cut notch. Use protective gloves, and throw them away at the end of the operation.
- Read carefully the instructions, the warnings and the SDS before use

Certificates:

- Class IIa-Medical Device
CE 0425
- The product has been tested to be effective according to the standards:
EN 13727, EN 13624

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SANITIZING ANTIFOAM TABLETS FOR DENTAL ASPIRATORS

Application: single-surgery and centralized suction systems

When to use: Every day, after cleaning and disinfection.

How to use: At the end or at the beginning of every working day, after cleaning, place 1 or 2 tablets in the first filter: when aspirated, the liquid will take up the right quantity of sanitizing antifoam product.

Range of action: The effectiveness lasts as an average a full working day. Check every 3/4 hours that the tablet has not melted and place a new one, if necessary

- Antifoam action
- Sanitizing-antibacterial

Precautions:

- Do not remove the protective film: it dissolves in the aspirated liquid.
- For small filters: remove the film, then break the tablet in half by pressing along the pre-cut notch. Use protective gloves and throw them away at the end of the operation
- Read carefully the instructions, the warnings and the SDS before use.

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HYGIENE AND DISINFECTION PROGRAM FOR DENTAL ASPIRATORS

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You can obtain a thorough cleaning of the aspiration system with:

- "Pulse Cleaner" creating the necessary turbulence.
- "Puli-Jet Plus New" disinfectant-detergent.
- Antifoam tablets, inhibiting foam formation and reducing the bacterial rate.
- "Pulse Cleaner" is a new turbulence-maker. To prepare the Puli-Jet Plus New solution, pour the concentrated product in warm water (+50 °C) and then place the terminals on the pistons. The cleansing aspiration will start without any further operation. The solution left in the container can be used to sanitize the spittoon bowl. Do not rinse the aspirator; do not turn on the spittoon-rinsing tap; Puli-Jet Plus New will go on disinfecting and dissolving blood and mucus all the night long.

ANTISCHUMOGENO DISINFETTANTE DISINFECTANT ANTIFOAM TABLETS - DISINFETTANTE PER ASPIRATORI DENTALI DISINFECTANT ANTIFOAM TABLETS FOR DENTAL ASPIRATORS.

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HOW IS IT WE LEAD IN OUR FIELD, WHEN WE COST LESS THAN THE ALTERNATIVES? THIS IS HOW:

Constant research: this enables us to apply the latest technology to all of our products and solutions.

We enhance performance: electronic and information technology enable us to enhance the performance and reliability of our products.

We reduce costs: less maintenance and lower energy costs mean that we are always the most economical on a cost-benefit analysis.

We reduce environmental impact: we save 50% on raw materials, and allow you to save between 30% and 50% on electrical consumption.

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